**Limited Two-Year Warranty**

Simple Diagnostics guarantees that its Digital Blood Pressure Monitor will be free from manufacturing defects under normal use for as long as the original purchaser owns the product. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the Digital Blood Pressure Monitor. This warranty is voided if the Digital Blood Pressure Monitor is misused or abused in any manner.

If the Digital Blood Pressure Monitor fails to operate during the time the original purchaser owns it, return it postage prepaid along with \$5.00 for return shipping and insurance to Simple Diagnostics, Attn: Repair Department, PO Box 128, Williston Park, NY 11596. When returning a product, please also include a letter with your name, address, phone number and brief description of the specific problem. Please carefully package the product to avoid any damage that may occur while in the transit. Packaging insurance with returned receipt is recommended. Simple will repair or replace the defective unit at the option of Simple Diagnostics. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

**ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT.** Some states do not allow limitations or how long an implied warranty lasts, so the above limitation may not apply to you. This warranty constitutes the warrantor's only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. **Simple will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others.** Simple shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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Simple Diagnostics

Blood pressure is measured in millimeters (mm) of mercury (Hg) and is generally recorded with the systolic pressure (120) listed first and the diastolic pressure (80) listed second. The numbers are typically separated by a slash mark (/) as shown.

Both pressure readings, the SYSTOLIC and DIASTOLIC, are necessary for a physician to evaluate the status of a patient's blood pressure.

Please contact your physician for specific information regarding your own blood pressure.

**What Influences Blood Pressure?**

Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure. Influences such as sleep or relaxation decrease blood pressure, while anxiety or exercise increase blood pressure.

**WHO Blood Pressure Classifications**

Standards for assessment of high or low blood pressure have been established by the World Health Organization (WHO) as shown on the following chart:

CATEGORY	SYSTOLIC (mmHg)	DIASTOLIC (mmHg)	COLOR INDICATOR
Optimal	<120	<80	GREEN
Normal	120-129	80-84	GREEN
High - Normal	130-139	85-89	GREEN
Stage 1 - Mild	140-159	90-99	YELLOW
Borderline Hypertension	160-179	100-109	ORANGE
Stage 2 - Moderate Hypertension	≥180	≥110	RED
Stage 3 - Severe Hypertension	≥140	<90	YELLOW
Isolated Systolic Hypertension	140-159	<90	ORANGE
	160-179	<90	RED
	≥180	<90	RED

This chart is only a general guideline. Contact your physician or trained healthcare professional to determine your NORMAL blood pressure.

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**Index**

Introduction	3
Product Identification & Caution	4
Why Monitor Your Blood Pressure at Home?	5
General Blood Pressure Information	5-7
Important Information Before Use	7
World Health Organization (WHO) Indicator	8
Irregular Heartbeat Detection	9
Battery Installation/Replacement	10
Setting the Date/Time	11
Voice Language Setting	12
Applying Your Blood Pressure Cuff	13
Taking Your Blood Pressure Reading	14-15
Recalling Measurements in Memory	15
Description of Display Symbols	16
Troubleshooting	16
Care and Maintenance	17
Product Specifications	17
Sample Blood Pressure Log	18

**Introduction**

To achieve the maximum benefit from your blood pressure monitor, we recommend that you first consult with your physician or trained healthcare professional.

Thank you for purchasing a Simple Automatic Digital Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

The method of measurement that your Automatic Monitor uses is called the oscillometric method. The monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor easy to use.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

**Battery Installation/Replacement**

1. Press down on tab and lift the cover.
  2. Insert or replace 4 "AA" batteries into the compartment, matching the indicated polarity symbols, Fig. 1.
  3. To close, match up and connect the bottom hooks of the cover, then push the top end.
- NOTE:** Replace the batteries when the Low Battery symbol appears on the display or when the display remains blank when the POWER button is pressed.
4. It is recommended to remove the batteries if the unit will not be used for an extended period of time.

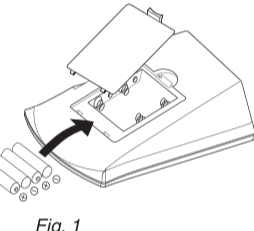
Fig. 1

**AC Power Adapter (optional)**  
 An AC adapter is available separately for battery-free operation. The AC adapter plugs into the side of the unit as shown, Fig. 2. Please remember to practice electrical safety precautions while using the adapter.

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Fig. 2

**Variations in Blood Pressure**

Blood pressure is influenced by many factors and can change from moment to moment. Normally, blood pressure is lowest during sleeping periods and rises during the day. The graph below represents variations in blood pressure shown over a day with measurements taken every 5 minutes.

The dotted line represents the sleep period. The rise in blood pressure at 1 p.m. (A in the graph) corresponds to a stressful occurrence and at 7 p.m. (B in the graph) a period of exercise.

**Important Information Before Use**

1. Blood pressure readings should be interpreted by a physician or trained healthcare professional, who is familiar with your medical history.
2. Perform your measurement in a quiet place. You should be seated and relaxed.
3. Avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading. If you are exhibiting signs of stress, avoid taking your measurement until the feeling subsides.
4. Rest 15 minutes prior to taking a reading.
5. Remove any constrictive clothing or jewelry that may interfere with the cuff placement.
6. Keep the monitor stable during measurements. Remain still; do not talk during measurements.
7. Record your daily readings on the chart provided.
8. Take your readings at the same time, each day (or as recommended by your physician).
9. Wait a minimum of 15 minutes between readings. The wait time may vary depending on individual physiological characteristics.
10. This device is intended for adult use.
11. The inflation or deflation process can be stopped by pressing the POWER button.

**Taking Your Blood Pressure Reading**

Proceed only after reading the previous sections of this manual.

1. Position the monitor on a flat, stable surface with the digital display panel in view. Insert the cuff tubing connector into the left side of the monitor, Fig. 1.
2. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig. 2. Relax your left hand.
3. Press the POWER button to turn the unit on and select the memory bank you want your reading to be stored in by pressing the MEMORY button to alternate between bank 1 or 2. Press the POWER button to confirm your selection.

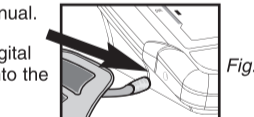
Fig. 1

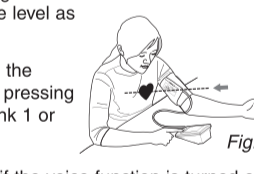
Fig. 2

**NOTA:** The monitor will begin to guide you with voice prompts if the voice function is turned on. This unit will run a self-test, Fig. 3, and show the last measured reading, Fig. 4, or if there is no previous measurement, "0" as shown in Fig. 5.

5. When the display appears as shown, Fig. 6 the monitor is ready and will begin to guide you with voice prompts if the voice function is on. The cuff will automatically inflate to begin.
6. The display will show the increasing pressure in the cuff. Along with the measurement progress on the left side of the screen.
7. The cuff will automatically inflate to approximately 190 mmHg or 40 mmHg higher than the last systolic pressure stored in memory.
8. Once the cuff inflates to the appropriate pressure level, the measurement will begin.
9. The unit will begin to automatically deflate and the numbers appearing on the display will decrease.
10. When all the air is deflated, the measurement is complete. The unit will display your blood pressure (systolic and diastolic) measurements and pulse rate, Fig. 7.

With the voice function on (L1 or L2), the monitor will say the measurement with the language you selected according to the classification standard of WHO (World Health Organization).

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**Setting the Date/Time**

**NOTE:** When the unit is off, the date and time will still appear at the top of the screen.

1. When the unit is off, press and release both the POWER and MEMORY button at the same time. The month will begin to flash, Fig. 1.
2. Press the MEMORY button to change until the desired number appears.

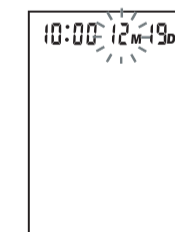
Fig. 1

**NOTE:** To scroll through the numbers quicker, hold down the MEMORY button.

3. Press the POWER button to set the entry.
4. The date, Fig. 2, followed by the hour and minutes will blink. Repeat steps 2 and 3 to adjust the date/time setting.
5. After setting the minutes, the unit will automatically exit out of the date/time setting mode, Fig. 3.

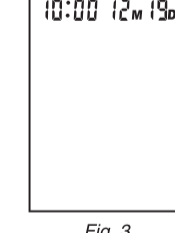
Fig. 2

**NOTE:** The time is expressed in 24hr military time. The chart below explains the time conversion.

Regular Time (12 hours)	Military Time (24 hours)	Regular Time (12 hours)	Military Time (24 hours)
12 am	0	12 pm	12
1 am	1	1 pm	13
2 am	2	2 pm	14
3 am	3	3 pm	15
4 am	4	4 pm	16
5 am	5	5 pm	17
6 am	6	6 pm	18
7 am	7	7 pm	19
8 am	8	8 pm	20
9 am	9	9 pm	21
10 am	10	10 pm	22
11 am	11	11 pm	23

**Sample Blood Pressure Log**

Name: \_\_\_\_\_ Age: \_\_\_\_\_ Weight: \_\_\_\_\_

Date	1/2	1/2	1/3
Time	7:00 AM	9:30 AM	7:30 AM
mmHg	130	125	120
	84	90	78
Pulse	68	71	69
Body Condition	Headache in AM		Tired, Restless Night

**MANUAL DE INSTRUCCIONES**