
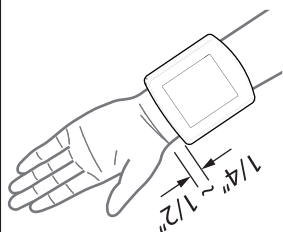
IMPORTANT!! You can stop the inflation or deflation process of the cuff at any time by pressing the POWER button.
Please read the Guidebook for this monitor prior to using.

- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; therefore, even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

PLEASE NOTE

- 1) Resting period – **A 15 minute resting period is needed to stabilize the body prior to taking a reading.** No eating, drinking or smoking while resting.
- 2) Cuff placement – Remove all jewelry and wrap the cuff snugly around your left wrist. **The cuff's top edge should be approximately 1/4"-1/2" from the bend in your wrist (see diagram).** The display screen and the palm should be facing the same direction. Make sure the top and bottom edges of the cuff are tightened evenly around your wrist.
- 3) Arm placement – Rest your elbow on the table and **raise your hand so that the cuff is at the same level as your heart.** A cuff that is below or above heart level will cause an inaccurate reading.
- 4) Press the POWER button to turn the unit on. Select the memory bank you want your reading to be stored in by pressing the MEMORY button to alternate between memory bank 1 or 2. Press the POWER button to confirm your selection. The cuff will automatically inflate. Do not move and remain quiet during inflation.
- 5) Memory feature – Press the MEMORY button and select the appropriate user bank by pressing the POWER button. Press the MEMORY button again to view previously stored readings (see page 16). Up to 120 measurements (60 per memory bank), including time and date, can be stored. See the enclosed Guidebook for further details about setting the time and date feature, irregular Heartbeat Detection and the WHO Indicator.
- 6) Avoid movement – **Eating, smoking or talking during the unit's operation will cause errors in readings.**
- 7) Avoid continuous readings – **A 15 minute rest period is needed between readings.** This allows the blood vessels to return to normal.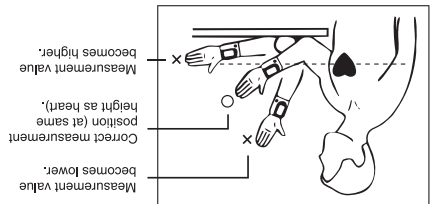

NOTE: This monitor is designed for a wrist circumference of 5-1/2" – 7-5/8" (14 - 19.5 cm).

The monitor requires 2 AAA alkaline batteries (not included).
Set the date and time before using this monitor (see page 11).

See the enclosed Guidebook for further details and instructions.

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